


Rapid Refresh® | Privacy Curtain Installation Kit Instructions

STEP 1:

Remove the old curtain from the hooks.

STEP 2:

Attach the D-ring at the end of Rapid Refresh rod to the curtain hook.

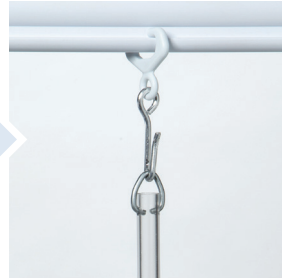



STEP 3:

Crimp the hooks closed to ensure the rods stay connected and do not fall off.




STEP 4:

Make sure the rods are not crisscrossed and all black dots are facing in the same direction. Insert curtain key from new curtain package.

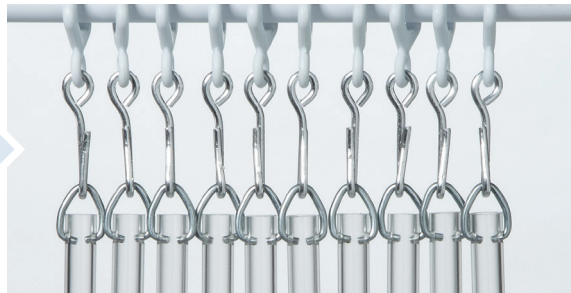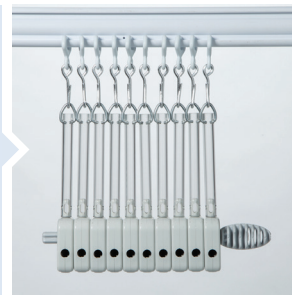

STEP 5:

Attach the Rapid Refresh Privacy Curtain, by lining up the square hangers on the top of the curtain panel with the bottom of the curtain rod adapters. Make sure all hangers are securely inserted into the bottom of the adapter. Your curtain now is secured to the rod adapters.

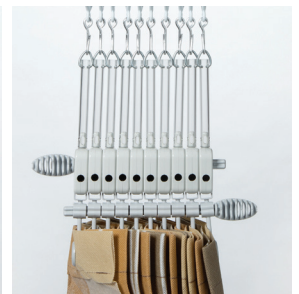

STEP 6:

Remove both curtain keys and open the curtain.



FOR MORE INFORMATION:

info.ICPMedical@teamtech.com

| 1-800-405-3044

| www.icpmedical.com