

Rapid Refresh™ | Privacy Curtains

Rapid Refresh™ Curtains can be changed in as little as 30 seconds. The patented release mechanism allows the curtain to be changed without the need for a ladder reducing the threat of falls, increasing the speed of a room cleaning and increasing compliance.



1

Open new curtain package at perforation.



2

Remove loose key from package.



3

Gather old curtain and insert key through upper holes.




4

Turn key toward you to release curtain.




5

Discard old curtain.

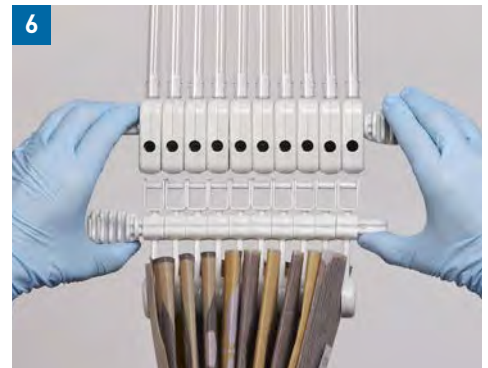


6

Line up new curtain with rod assembly and click into place.

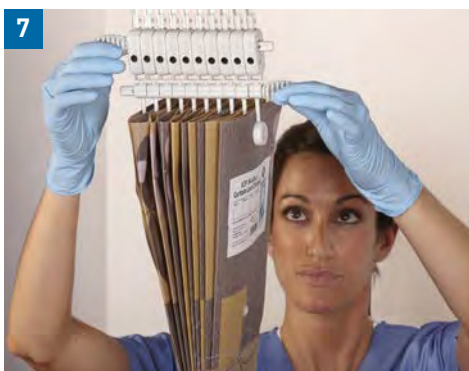

7

Remove and discard both keys.

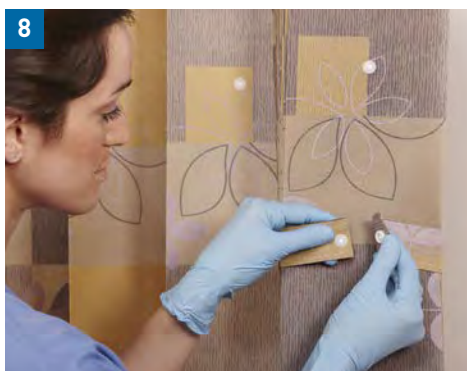

8

Remove plastic sleeve and band from new curtain.



9

Open new curtain and activate Timestrip®.

Timestrip® is a registered trademark of Timestrip UK Ltd.