

DENSO Health Scanner ICP-SC1-QB

Basic Care & Maintenance

Dust and other foreign matter on the reading window can impede code input, regularly check for debris and remove it as the usage environment warrants.

- To clean the window, first blow the dust away with an airbrush. Then, wipe the reading window with a cotton swab or similar soft cloth.
- If sand or hard particles have accumulated, never rub the reading window; doing so will scratch or damage it. Blow the particles away with an airbrush or soft brush.



Care & Maintenance with disinfectant or soap

1. Wet a soft cloth with our tested disinfectants or soaps.
2. Wipe scanner body and reading window
3. Wipe remaining disinfectant or soap off scanner body and reading window.

(CAUTION)

- When using liquid or spray type of disinfectants or soaps, do NOT dip or directly spray on scanner.
- Do not soak scanner in disinfectant, soap or water.
- In case of product discoloration, stop using disinfectant immediately.

Our Tested Disinfectants and Soaps

We have confirmed no appearance, function and performance damage on the scanner based on a once a day wipe test on the following products.

- 91% Isopropyl Alcohol Solution
- 0.5% Hydrogen Peroxide Solution
- 3% Hydrogen Peroxide Solution
- Sani-Cloth® Bleach Wipes [PDI]
- Sani-Cloth® Plus Germicidal Wipes [PDI]
- Sani-Cloth® HB Germicidal Wipes [PDI]
- Sani-Cloth® PDI AF3 Wipes [PDI]
- Super Sani-Cloth® Wipes [PDI]
- Disinfecting Wipes (Bleach-free)[Clorox]
- Formula 409 Multi-Surface [Clorox]
- Dispatch Wipes [Clorox]
- MetriCide 28 Day Solution (2.5% Glutaraldehyde) [Metrex]
- CaviWipesXL Disinfecting Towelettes [Metrex]
- Cidex OPA Solution [Advanced Sterization Products]
- Oxivir Tb Wipes [Diversey]
- Virex II 256 Disinfectant Cleaner [Diversey]
- Hepacide Quat II [Spartan]
- Windex original Spray [SC Johnson]
- Windex Multi-Surface Anti-Bacterial Spray [SC Johnson]

