

Bootie Butler® Removal


ICP Medical®
by Cordica Medical

The Bootie Butler® Shoe Cover System makes removing shoe covers a simple, safer and money-saving process.

To remove shoe covers:

Para quitar las cubiertas de zapatos:


01/

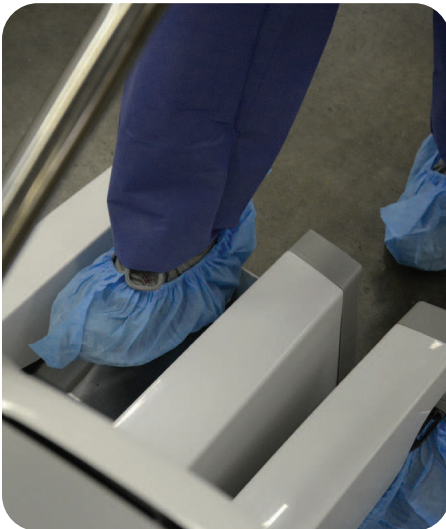

Place one foot at a time over opening on remover.

Coloque un pie a la vez sobre la abertura del removedor.

02/

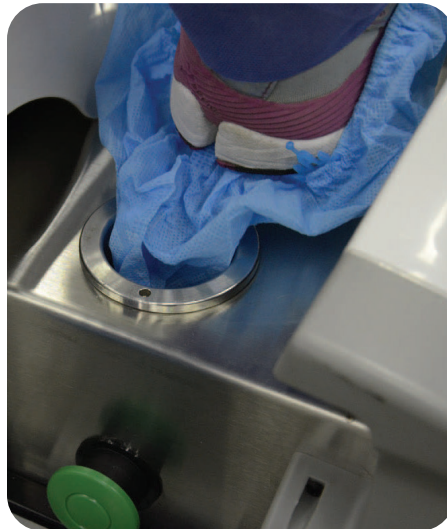

With toe raised to a 45 degree angle, align heel of shoe cover with suction portal.

Con la punta elevada en un ángulo de 45 grados, alinee el talón de la cubierta del zapato con el portal de succión.

03/



Slowly slide foot backward as shoe cover is suctioned into portal and entire shoe cover is removed.

Deslice lentamente el pie hacia atrás mientras se succiona la cubierta del zapato hacia el portal y se retira toda la cubierta del zapato.